# EN DeVilbiss® PulmoNeb™ LT Compressor Nebulizer System Instruction Guide

## Model 3655LT Series

**CAUTION**—Federal (U.S.A.) law restricts this device to sale by, or on the order of a physician.

Latex Free.

3655LT Series Intertek ETL Certified to UL Std. 60601-1

### IEC SYMBOLS

- ON
- OFF
- Attention, consult instruction guide.
- Alternating Current
- Type BF Applied Part
- Double Insulated
- This device contains electrical and/or electronic equipment that must be recycled per EC Directive 2002/96/EC - Waste Electrical and Electronic Equipment (WEEE)

### READ ALL INSTRUCTIONS BEFORE USING.

## ▲ DANGER

#### To reduce the risk of electrocution:

- Always unplug this product immediately after using.
- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink or other water source.
- Do not reach for a product that has fallen into water. Unplug immediately.
- DeVilbiss compressor/nebulizer must be operated on the specified power source to avoid risk of electric shock and damage to the compressor.
- Do not submerge in water; doing so will result in compressor damage.
- Electric shock hazard. Do not remove compressor cabinet. All disassembly and maintenance must be done by a qualified DeVilbiss provider.

## ▲ WARNING

#### To reduce the risk of burns, electrocution, fire or injury to persons:

- The product should never be left unattended when plugged in.

- Close supervision is necessary when this product is used by, on, or near children or invalids.
- Use this product only for its intended use as described in this guide. Do not use attachments not recommended by the manufacturer.
- Never operate this product if:
  - It has a damaged power cord or plug.
  - It is not working properly.
  - It has been dropped or damaged.
  - It has been dropped into water.
 Return the product to an authorized DeVilbiss service center for examination and repair.
- Keep the power cord away from heated surfaces.
- Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of debris.**
- Never use while drowsy or asleep.
- Never drop or insert any object into any opening or hose.
- Do not use outdoors. This product is for household use only.
- Do not use in an oxygen enriched environment.
- Do not use this product with a DC to AC converter, or with any AC voltage and/or frequency other than specified.
- Do not operate unit in a backpack or carrying bag.
- Use caution when plugging in and unplugging device. Do not allow jewelry or other metal items to contact plug.
- This product contains small parts that may represent a choking hazard.**
- To prevent possible risk of infection from contaminated medication, cleaning of the nebulizer is recommended after each aerosol treatment. Daily disinfecting is recommended.
- To prevent possible risk of infection from contaminated cleaning solutions, always prepare fresh solution for each cleaning cycle and discard solution after each use.

**NOTE**—This unit employs a temperature and/or current limiting device that will shut off the entire product should a malfunction occur or in the event of product misuse, abuse, or abnormal operation.


**NOTE**—The exterior surfaces of the unit and the air outlet connector may become slightly warm to the touch during use. Warm air can also be felt exiting the vents on the sides of the case of the unit. This is normal.

## SAVE THESE INSTRUCTIONS

### STATEMENT OF INTENDED USE

The DeVilbiss compressor nebulizer includes an AC powered air compressor that provides a source of compressed air for home health care use. The compressor is used in conjunction with a jet (pneumatic) nebulizer to convert certain inhalable drugs into an aerosol form for inhalation by a patient. The device can be used with adult or pediatric patients.

### IMPORTANT PARTS OF YOUR DEVILBISS 3655LT SERIES COMPRESSOR AND NEBULIZER

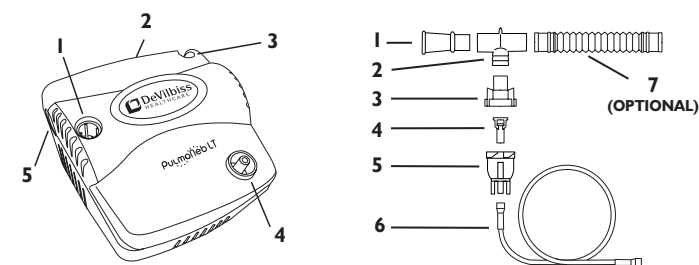#### COMPRESSOR

- Filter (non-washable)
- Carrying handle
- Nebulizer holder
- Air-outlet connector
- Power switch

#### NEBULIZER

- Mouthpiece
- T-piece
- Cap
- Baffle
- Medication Cup
- Nebulizer Air-Inlet Connector
- Corrugated Tubing (disposable neb only\*)

\*Note: Depending on model purchased, your unit may include a disposable or reusable nebulizer, or both.

### HOW TO OPERATE YOUR COMPRESSOR

**NOTE**—Before initial operation, clean nebulizer.

- Ensure the power switch is in the “O” off position.
  - Place compressor on a level, sturdy surface. Never block air vents. Never place compressor on a soft surface, such as a patient’s lap, or a bed or couch, where the air openings may be blocked.
  - Plug power cord into wall outlet.
  - Wash hands.
  - Connect one end of the nebulizer tubing to the compressor air-outlet connector.
- NOTE**—During high-humidity weather conditions, condensation (water build up) can occur in the nebulizer tubing.
- Assemble nebulizer as shown. If using an aerosol mask, insert the bottom part of the mask directly into the top of the nebulizer cap.
  - Press the power switch to start compressor.
  - Place the mouthpiece between teeth. With mouth closed, inhale deeply and slowly through mouth as aerosol begins to flow, then exhale slowly through the mouthpiece. If treatment needs to be interrupted, simply press power switch and place the nebulizer into nebulizer holder.
  - If an aerosol mask is used, place the mask over mouth and nose. As aerosol begins to flow, inhale deeply and slowly through mouth, then exhale slowly.
  - When treatment is complete, turn unit off. Unplug unit from wall outlet.

### NEBULIZER CLEANING

- Disassemble nebulizer.
- Wash all items, except tubing, in a hot water/dishwashing detergent solution. Rinse under hot tap water for 30 seconds to remove detergent residue. Allow to air dry.

#### Disinfect Daily:

- Using a clean bowl, soak items in three parts hot water to one part white vinegar for 30 minutes OR use a medical bacterial-germicidal disinfectant. Be sure to follow manufacturer’s instructions carefully.
- With clean hands, remove items, rinse under hot tap water, and air dry. Store in a zip-lock bag.

**NOTE**—Do not towel dry nebulizer parts; this could cause contamination.

- Keep the outer surface of the tubing dust-free by wiping regularly. Nebulizer tubing does not have to be washed because only filtered air passes through it.

**NOTE**—Replace your DeVilbiss primary reusable nebulizer every 6 months. Disposable nebulizer (often used as a backup) should be replaced at least every 15 days. To ensure consistent operating specifications, replace with DeVilbiss nebulizer.

### COMPRESSOR CLEANING

- With power switch in the “O” off position, unplug power cord from wall outlet.
- Wipe outside of the compressor cabinet with a clean damp cloth every few days to keep dust-free.

#### Filter Change

- Filter should be changed every 6 months or sooner if filter discolors.
- Remove filter by grasping it firmly and pulling out of the unit. Discard old filter.
- Replace with a new DeVilbiss filter (part #3655LT-601). Additional filters should be purchased from your DeVilbiss provider.

**CAUTION**—Do not reuse or wash filter or substitute any other material for a DeVilbiss air-inlet filter; it will result in compressor damage.

### MAINTENANCE

All maintenance must be performed by a qualified DeVilbiss provider.

### SPECIFICATIONS

Size	5.75" W x 4.0" H x 6.0" D (14.6 cm x 10.2 cm x 15.2 cm)
Weight	2.75 lbs. (1.25 kg)
Electrical Requirements	2.5 amps, 115 VAC (±10%) 60 Hz

Sound Level	56 dBA
Power Consumption	140 watts maximum
Maximum Compressor Pressure	30 psig or greater (206 kPa)*
Compressor Free Air Flow	9 lpm or greater*
Operating Temperature Range	+40° to +104°F (+5 to +40°C)
Operating Humidity	up to 95% non-condensing
Storage/Transport Temperature Range	-40° to +158°F (-40° to +70°C)
Storage/Transport Humidity	up to 95% non-condensing
Supplied Nebulizer Capacity	6 ml (cc)
Supplied Nebulizer Nebulization Rate	0.15 ml/min or greater*
Supplied Nebulizer MMAD	5 microns or less**

\*Conditions may vary based on altitude above sea level, barometric pressure, and temperature.

\*\*Value determined with minimum flow rate through the nebulizer (20°C, sea level).

There are no hazardous materials used in the construction of this unit, therefore, there are no known risks associated with disposal of the equipment.

This product complies with the following electromagnetic compatibility standard: IEC 60601-1-2:2nd Edition.

### Equipment Classifications:

Protection against electric shock: ..... Class II  
 Degree of protection against electric shock: ..... Type BF  
 Equipment not suitable for use in the presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.  
 Mode of operation: .....Continuous operation

### FIVE-YEAR LIMITED WARRANTY ON COMPRESSOR PARTS

All DeVilbiss Compressor models are warranted to be free from defective workmanship and material for a period of five years from the date of purchase. Any defective part(s) will be repaired or replaced at DeVilbiss' option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your authorized DeVilbiss provider or DeVilbiss Service Department at 800-338-1988 (814-443-4881) for instructions. **NOTE**— This warranty does not cover providing a loaner compressor, compensating for costs incurred in compressor rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY, IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

DeVilbiss Healthcare LLC • 100 DeVilbiss Drive • Somerset, PA 15501-2125 • USA  
 800-338-1988 • 814-443-4881 • www.DeVilbissHealthcare.com

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