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TABLE OF CONTENTS

Important Safeguards	EN-2
Intended Use	EN-3
Important Parts	EN-3
Setup and Operation	
Headgear Assembly	EN-4
Applying the Mask	EN-5
Circuit Connection	EN-5
Removing the Mask	EN-6
Adjustable Vent	EN-6
Oxygen/Pressure Port	EN-6
Care And Cleaning Instructions	EN-6
Reassemble Mask	EN-7
Replacement Parts	EN-7
Specifications	EN-7
Warranty	EN-8

IMPORTANT SAFEGUARDS

Basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

WARNING Important safety information for hazards that might cause serious injury or death.


CAUTION Information for preventing damage to the product.

NOTE Information to which you should pay special attention.

PLEASE READ ALL INSTRUCTIONS BEFORE USING.

WARNING

- This CPAP mask should be used only with CPAP systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP system is turned on and operating properly. The vent hole(s) associated with the mask should never be blocked.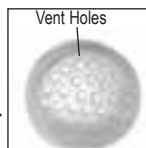

Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP unit is turned on and functioning properly, new air from the unit flushes the exhaled air out through the mask vent hole(s). However, when the CPAP unit is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This warning applies to most models of CPAP systems.

- **CAUTION**—At low CPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- **NOTE**—At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP systems.
- If oxygen is used with the CPAP system, the oxygen flow must be turned off when the CPAP machine is not operating.

Explanation of the Warning: When the CPAP device is not in operation and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire. This warning applies to most types of CPAP systems.

SAVE THESE INSTRUCTIONS

INTENDED USE (FOR SINGLE-PATIENT USE)

This product is intended to be used with nasal continuous positive airway pressure (CPAP), auto-adjusting CPAP, and bilevel systems for the treatment of adult (>30Kg) Obstructive Sleep Apnea (OSA). It is to be used in home and clinical environments by or on the order of a physician.

NOTE—The FlexSet Mask can be used with auto-adjusting PAP devices which depend on patient flow signals or flow-based snoring signals for the determination of pressure adjustment.

IMPORTANT PARTS OF YOUR DEVILBISS FLEXSET MASK (FIGURE A)

1. Headgear Buckle
2. Forehead Pad
3. Forehead Rest
4. Upper Mask Bridge Levers
5. Mask Bridge
6. Lower Mask Bridge Levers
7. Oxygen/Pressure Port Cover
8. Elbow Lock
9. Mask Cushion (silicone/gel/ComfortTouch™ options)
10. Adjustable Vent
11. Mask Body
12. Mask Tusks
13. Mask Elbow
14. Quick Connect/Swivel Adapter
15. Over-the-Head Tubing Strap (optional)
16. Extra Forehead Pad & Velcro Hook Strap (optional)

Tubing Options (Figures B & C)

The DeVilbiss FlexSet Mask comes fully assembled. The mask provides you with two options for orienting the air supply tubing. Review Figures B & C to choose the best option for you.

- One option allows the air supply tubing to rotate freely 360 degrees.
- One option routes the air supply tubing over the top of your head.

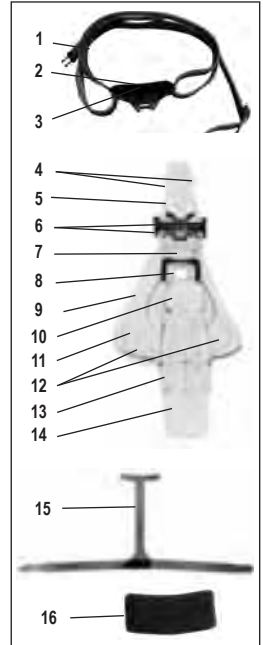
Figure A


Figure B - 360° Tubing Option


Figure C - Over-the-Head Tubing Option

SET-UP & OPERATION


Headgear Assembly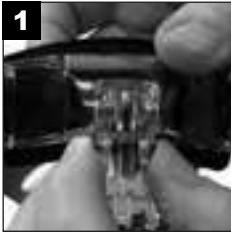

1 Press both Upper Mask Bridge Levers and remove mask from headgear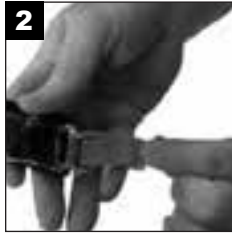

2 Loosen upper Velcro straps on headgear

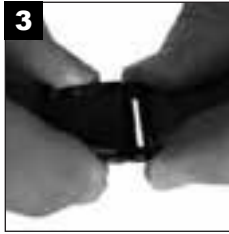
3 Unclasp headgear buckle.

4 Place headgear on your head like a headband with Forehead Rest centered on your forehead.

5 Gently tighten both upper Velcro straps so headgear is comfortably snug. Do not overtighten.

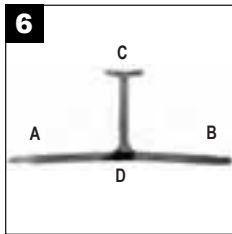6 **Only if using "Over-the-Head" Tubing Option as shown in Fig. C:** Remove headgear and apply over-the-head strap by attaching C to the back of the headgear and A & B over the sides. Place headgear back on and adjust as necessary until it is comfortably snug.

7 Connect headgear buckle and place lower strap over your chin.

8 Adjust so lower strap fits loosely across chin. **NOTE**—Headgear buckle position on cheek can be changed by adjusting bottom straps.

Applying the Mask

Gently push Mask Bridge upward into opening on Forehead Rest until it snaps into place.

Pull lower headgear strap over mask "tusks" and ensure strap is flat against mask body and your cheek. If needed, adjust lower Velcro straps for a tight but comfortable fit.

Mask should be parallel with your face. Adjust mask angle either backward or forward by pushing Lower Mask Bridge Levers upward to disengage mask bridge teeth while holding body of mask steady with other hand.

Over-the Head


Over the Head

Attach CPAP tubing to 15 mm tubing w/swivel adapter on FlexSet mask.

Over-the-Head Tubing Option (Fig. C): Run air supply over back of head and down into Mask Elbow and secure tubing in Velcro loop on top of headgear.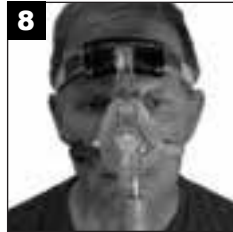

Place mask on your face and turn on your CPAP device and verify airflow through air supply tubing.

Circuit Connection

Adjust upper and lower Velcro straps as needed to eliminate leaks and optimize comfort. If needed, mask angle can also be adjusted by using Lower Mask Bridge Levers.

CAUTION—Do not overtighten your headgear! Excessive tension will cause discomfort during the night. Reposition mask on your face and adjust your headgear tension.

Removing the Mask

Remove lower strap from mask "tusks" and then disconnect mask from headgear by pressing Upper Mask Bridge Levers. **NOTE**—If using "Over-the-Head" Tubing Option (Fig. C): Also release tubing from Velcro loop

If you prefer to leave the mask attached, simply pull off Quick Connect from Mask Elbow. **NOTE**—If using "Over-the-Head" Tubing Option (Fig. C): Also release tubing from Velcro loop.


Adjustable Vent

Grasp adjustable vent and rotate to desired position to change the direction of airflow exiting exhaust port.

Oxygen/Pressure Port

This is for monitoring pressure during a sleep study or adding oxygen if prescribed by your physician.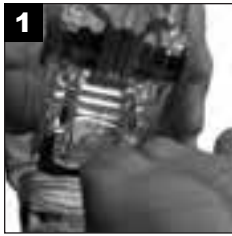

Remove the oxygen/pressure port cover and secure your tubing to this port.

WARNING

Oxygen supports combustion. To avoid possible physical injury, do not smoke while using your CPAP with supplemental oxygen. Do not use near hot sparking objects or open sources of flame.

CARE AND CLEANING INSTRUCTIONS

Cleaning is recommended on a daily basis.

CAUTION—Do not use bleach, chlorine-, alcohol-, or aromatic-based cleaning solutions including scented oils. These solutions may cause hardening and reduce the life of the product. Do not place tubing in direct sunlight as it may harden and crack.

WARNING


CHOKING HAZARD. Contains small parts. Keep out of reach of children.

Disassemble the tubing, headgear, mask cushion, and forehead pad.

Wash all of the parts, including headgear, with warm water and mild soap. Rinse thoroughly with warm water.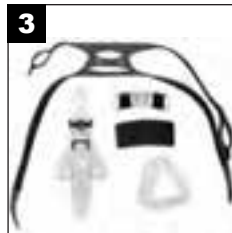

Allow parts to air dry. **NOTE**—Do not towel dry parts; this could cause contamination.

Weekly Cleaning (if desired): The FlexSet Mask is dishwasher safe if placed in top rack; small components should be secured. It is recommended that the mask be washed separately. Air dry before use.

REASSEMBLE MASK

Replace mask cushion on mask body by first aligning the top alignment mark on cushion with alignment arrow on mask body. Push the top area of cushion onto mask body, then continue attaching along sides and bottom until cushion is completely secured. **NOTE**—If necessary, adjust cushion position to eliminate wrinkles.

Reattach Forehead Pad to Forehead Rest and reassemble Headgear to Mask for your preferred configuration.

NOTE—For optimum mask performance, it is recommended that you replace your mask at least once a year. See your DeVilbiss equipment provider for replacement information.

WARNING

This nasal CPAP mask is intended for single-patient use only. The mask may be cleaned and reused on the same person.

REPLACEMENT PARTS

	MODEL	DESCRIPTION
Complete Mask	9354L	Large, Silicone
	9354D	Medium, Silicone
	9354S	Small, Silicone
	9354G	Medium, Gel
	9354GS	Small, Gel
	9354LR	Large, ComfortTouch Silicone
	9354DR	Medium, ComfortTouch Silicone
	9354SR	Small, ComfortTouch Silicone
	9354GR	Medium, ComfortTouch Gel
	9354GSR	Small, ComfortTouch Gel

	MODEL	DESCRIPTION
Cushion Replacement	9352L-601	Large, Silicone
	9352D-601	Medium, Silicone
	9352S-601	Small, Silicone
	9352G-601	Medium, Gel
	9352GS-601	Small, Gel
	9352LR-601	Large, ComfortTouch Silicone
	9352DR-601	Medium, ComfortTouch Silicone
	9352SR-601	Small, ComfortTouch Silicone
	9352GR-601	Medium, ComfortTouch Gel
	9352GSR-601	Small, ComfortTouch Gel
Forehead Pad & Rest	9353D-604	
Headgear	9353D-619	Includes Buckle & Over-The-Head Tubing Strap

SPECIFICATIONS


Mask Physical Dead Space

MODEL	DESCRIPTION	VOLUME (ML)
9354S/9354SR	Silicone Small	130
9354D/9354DR	Silicone Medium	133
9354L/9354LR	Silicone Large	141
9354GS/9354GSR	Gel Small	145
9354G/9354GR	Gel Medium	143

Mask Flow Resistance

FLOW RATE (L/MIN)	PRESSURE DROP (CMH ₀)
50	0.091
100	0.401

Pressure Flow Graph

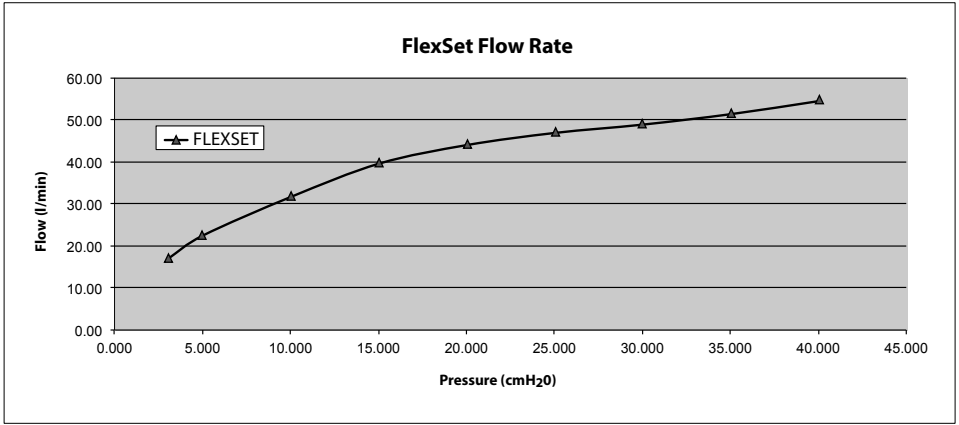Expected Service Life

- Mask Frame Components = 3 months
- Headgear = 6 months
- Mask Cushions = 1 month

WARRANTY

DeVilbiss FlexSet Mask is warranted to be free from defective workmanship and material(s) for a period of six months from date of purchase. Any defective part(s) will be repaired or replaced at DeVilbiss' option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your DeVilbiss provider or DeVilbiss Service Department for instructions:

U.S.A. 800-338-1988, 1-814-443-4881
Europe +49-621-178-98-230

NOTE—This warranty does not cover providing a loaner unit, compensating for costs incurred in rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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DeVilbiss® and FlexSet® are registered trademarks of DeVilbiss Healthcare. ComfortTouch™ is a trademark of DeVilbiss Healthcare.
© 2013 DeVilbiss Healthcare LLC. 01.13 All Rights Reserved.
DeVilbiss Healthcare Ltd
DeVilbiss Healthcare Pty. Limited
DeVilbiss Healthcare SAS
DeVilbiss Healthcare GmbH