



EasyFit Nasal Mask, Silicone

EasyFit Nasal Mask, Gel

Nasal Mask

Cleaning and Disinfection Instructions
for Multiple-Patient Use
(For Sleep Laboratory Applications only)

Caution:

Federal Law (U.S.A.) restricts this device to sale by or on the order of a physician.

Latex-free

Cleaning and Disinfecting Instructions

Before using the nasal mask for another patient, first subject it to appropriate hygiene measures. Disassemble the mask and clean it, then disinfect or sterilize all parts as described below.

Discard and replace mask components if they show signs of wear such as rips, tears or cracks.

When used for multiple patients, these instructions supersede the cleaning and disinfection instructions in the instructions for use provided with each mask.

See instructions for use included with each mask for Warnings and Cautions as well as for general directions for use and reordering information.

Procedures allowed

Caution!

The headgear is not intended for multiple patient use. To avoid patient cross infection always supply a new headgear when using the mask for another patient.

The following procedures are allowed:

Parts	Procedures	Manual Cleaning	Disinfecting*	Sterilizing (as an alternative to disinfecting)
Mask Cushion, Silicone		•	•	•
Mask Cushion, Gel		•	•	
Forehead Cushion, Silicone		•	•	
Forehead Cushion, Gel		•	•	
Oxygen Port Cap		•	•	•
Mask Frame		•	•	
Fine Adjustment		•	•	
Coarse Adjustment		•	•	
Elbow		•	•	
Rotating Sleeve		•	•	
Retaining Ring		•	•	
Headgear Clip		•	•	

Manual Cleaning	Immerse all mask components in clean warm water with mild hand dishwashing detergent. Use a piece of soft cloth to remove residue. To clean cavities use a soft brush. Carefully rinse the parts with clean water.
Disinfecting*	Clean manually first as described under "Manual Cleaning." Then clean in a diluted solution of disinfectant using mechanical means, e.g., a brush. After disinfecting, rinse the parts thoroughly with distilled water and dry them completely with a clean lint-free cloth before reassembling the mask.
Sterilizing (as an alternative to disinfecting)	Clean manually first as described under "Manual Cleaning." Then steam-sterilize at a temperature of 134 °C (273 °F), Minimum time: 3 minutes

* We recommend using undiluted high-level disinfectant Cidex OPA Solution (0.55% orthophthalaldehyde) only. Tolerance of this disinfectant has been proven. Make sure you carefully follow the manufacturer's directions.

After cleaning and disinfection

- Dry all parts thoroughly with a clean lint-free cloth.
 - Check components for residue or damage. Repeat procedures above if necessary. If parts are damaged, replace them.
 - Assemble the nasal mask as described in the instructions for use.
-

Manufactured by:

Weinmann
Geräte für Medizin GmbH+Co. KG
P.O. Box 54 02 68 · 22502 Hamburg
Germany
Phone +49/40/5 47 02-0
Fax +49/40/54 70 24 61
E-mail info@weinmann.de
Internet www.weinmann.de

Distributed by:

DeVilbiss Healthcare
100 DeVilbiss Drive
Somerset, Pennsylvania, 15501
USA
Phone 814.443.4881
Internet www.devilbisshealthcare.com

© 2009 DeVilbiss Healthcare
EasyFit™ is a trademark of DeVilbiss
Healthcare, a division of Sunrise Medical, HHG

DV67106 - 09.09