DEVILBISS®

DeVilbiss® FlexSet™ Mask

9354 Series Instruction Guide

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Sunrise Medical
Respiratory Products Division
100 DeVilbiss Drive
Somerset, Pennsylvania 15501
USA
814-443-4881

Sunrise Medical Canada, Inc.
237 Romina Drive, Unit 3
Concord, Ontario L4K 4V3
CANADA
905-660-2459

Sunrise Medical Ltd.
Sunrise Business Park
High Street
Wollaston, West Midlands
DY8 4PS ENGLAND
44-138-444-6688

Sunrise Medical Pty. Limited
15 Carrington Road, Unit 7
Castle Hill NSW 2154
AUSTRALIA
61-2-9899-3144

Sunrise Medical
Division Respiratoire
13 Rue de la Painguetterie
37390 Chateaux/Choisille
FRANCE
33-247-88-58-36

Sunrise Medical
DeVilbiss Produkte
Industriegebiet
69254 Malsch/Heidelberg
GERMANY
49-7253-980-460

Sunrise Medical Benelux
Pascalbaan 3
Nieuwegein 3439 MP
NETHERLANDS
31-3060-82-100

Sunrise Medical
Rehabstererit
1450 Nesoddtangen
NORWAY
47-66-96-38-00

IMPORTANT SAFEGUARDS

Basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

▲ WARNING Important safety information for hazards that might cause serious injury or death.

CAUTION Information for preventing damage to the product.

NOTE Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

▲ WARNING


- This CPAP mask should be used only with CPAP systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP system is turned on and operating properly. The vent hole(s) associated with the mask should never be blocked.
Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP unit is turned on and functioning properly, new air from the unit flushes the exhaled air out through the mask vent hole(s). However, when the CPAP unit is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This warning applies to most models of CPAP systems.
- **CAUTION**—At low CPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- **NOTE**—At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP systems.
- If oxygen is used with the CPAP system, the oxygen flow must be turned off when the CPAP machine is not operating.
Explanation of the Warning: When the CPAP device is not in operation and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire. This warning applies to most types of CPAP systems.

SAVE THESE INSTRUCTIONS.

MASK PHYSICAL DEAD SPACE

The physical dead space of the four models of the DeVilbiss FlexSet Mask is as follows:

- 9354D Silicone Standard 128 mL
- 9354S Silicone Shallow 129 mL
- 9354G Gel Standard 120 mL
- 9354GS Gel Shallow 118 mL

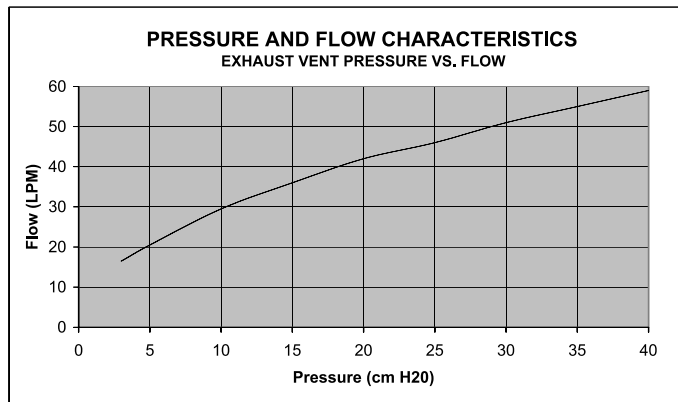PRESSURE FLOW GRAPH

USE INSTRUCTIONS (FOR SINGLE-PATIENT USE) – This product is intended to be used with nasal continuous positive airway pressure (CPAP), auto-adjusting CPAP, and bilevel systems for the treatment of adult (>30Kg) Obstructive Sleep Apnea (OSA). It is to be used in home and clinical environments by or on the order of a physician.

NOTE—The FlexSet Mask can be used with auto-adjusting PAP devices which depend on patient flow signals or flow-based snoring signals for the determination of pressure adjustment.

⚠ WARNING

CHOKING HAZARD. Contains small parts. Keep out of reach of children.

IMPORTANT PARTS OF YOUR DeVILBISS FLEXSET MASK (FIGURE 1)

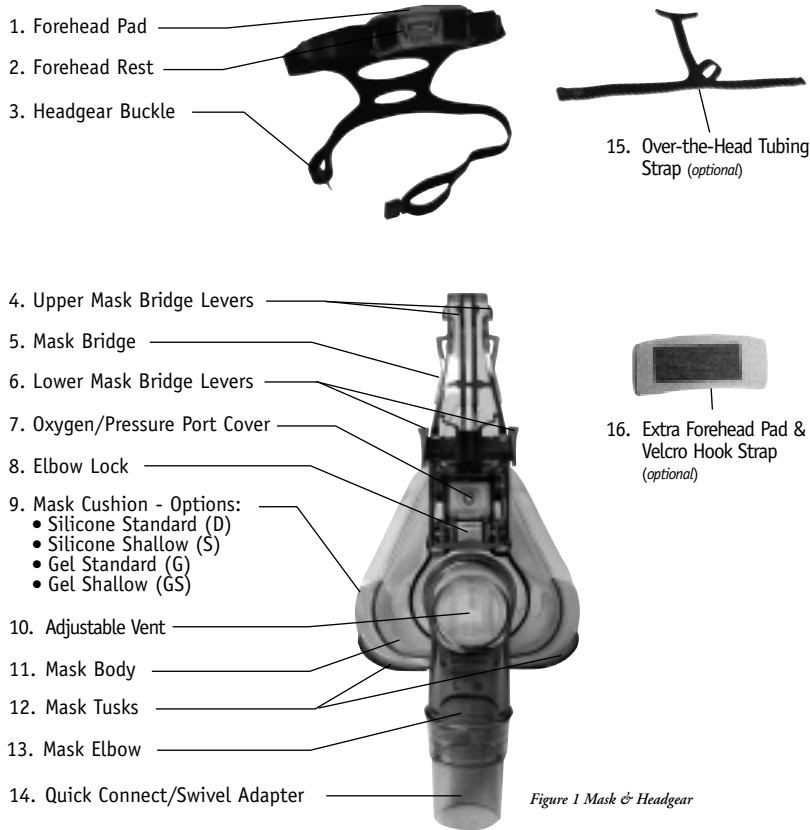
Figure 1 Mask & Headgear

The DeVilbiss FlexSet Mask is supplied fully assembled. The FlexSet Mask provides you with two options for orienting the air supply tubing. Review the illustrations (Figures 2 & 3) and choose the best option for you:

–One option allows you to let the air supply tubing rotate freely 360 degrees (page 7).

–The second option allows you to fix the air supply tubing over the top of your head (page 8).

Figure 2 - 360 Degree Tubing Option

Figure 3 - Over-the-Head Tubing Option

REPLACEMENT PARTS

- | | |
|--------------------------------------------------------------------------------------------------------|-----------|
| 1. Mask Cushion (9) X designates extension as follows: D=Standard; S=Shallow; G=Gel; GS=Gel Shallow | 9352X-601 |
| 2. Adjustable Vent (10)/Forehead Pad (1) & Rest (2) | 9353D-604 |
| 3. Headgear (includes buckle & over-the-head tubing strap) | 9353D-619 |

ALPHABETICAL PARTS CROSS REFERENCE

- | | |
|---------------------------------------|--------------------------------------------|
| Adjustable Vent (10) | Mask Bridge (5) |
| Elbow Lock (8) | Mask Cushion (9) |
| Forehead Pad (1) | Mask Elbow (13) |
| Forehead Pad & Velcro Hook Strap (16) | Mask Tusks (12) |
| Forehead Rest (2) | Over-the-Head Tubing Strap (optional) (15) |
| Headgear Buckle (3) | Oxygen/Pressure Port Cover (7) |
| Lower Mask Bridge Levers (6) | Quick Connect/Swivel Adapter (14) |
| Mask Body (11) | Upper Mask Bridge Levers (4) |

FAMILIARIZING YOURSELF WITH YOUR MASK (FIGURE 4)

- 1. PRESS** — Press the two Upper Mask Bridge Levers and remove the Mask from the Forehead Rest. This allows you to remove the Mask without removing the Headgear.
- 3. PULL** — In one hand, hold the body of the Mask and with the other hand, pull the two Lower Mask Bridge Levers upward from the top and move the mask bridge back and forth. This action permits the Mask to be adjusted to the proper angle of your nose.
- 5. PULL/PUSH** — While holding the Mask Elbow, pull off and push on the air supply tubing Quick-Connect. This allows you to remove the air supply tubing without removing the Mask.
- 7. PRESS** — Press the two tabs on the Headgear Buckle on the bottom strap of the Headgear to disconnect **Figure 4A**.

NOTE—This should remain disconnected until headgear is applied.

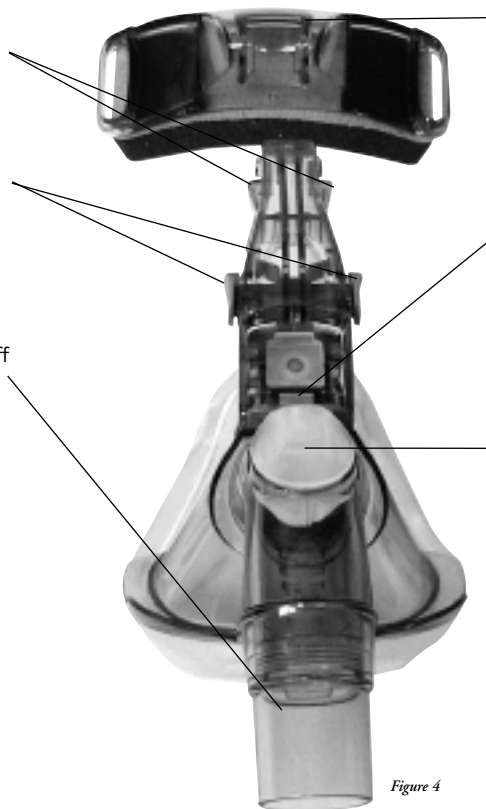
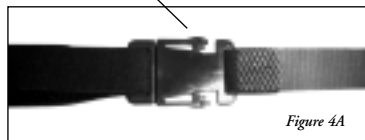
Figure 4

- 2. PUSH** — Gently push the Mask Bridge into the Forehead Rest – it is not necessary to press the Mask Bridge Levers when inserting the Mask Bridge into the Forehead Rest. Repeat this a few times until you are familiar with the action. Note that this piece swings freely to allow for proper fit to the angle of your forehead.
- 4. PULL** — Pull the Elbow Lock forward halfway and move the Mask Elbow to the upright position. Using your two index fingers on either side of the Elbow, pull the Elbow Lock down to lock the Mask Elbow into place. This allows you to orient the Headgear over the top of your head if desired. Push the Elbow Lock back into place to permit 360 degree rotation.
- 6. TURN** — While holding the Mask Elbow, turn the Adjustable Vent from one side to the other. This allows you to direct the flow of exhaled air away from yourself or your bed partner.

NOTE—The Headgear has an optional Over-The-Head Strap. This can be applied if you choose this option **Figure 3**.

NOTE—An optional Forehead Pad with Velcro Hook Strap is enclosed. It can be attached to the original Forehead Pad for added comfort.

Figure 5-360 Degree Rotation Option

MASK ASSEMBLY AND APPLICATION

360 Degree Tubing Rotation Option (refer to Figure 5)

This option allows the air supply tubing to rotate freely 360 degrees.

Applying the Headgear (360 Degree Option)

1. With the mask removed from the headgear, unlatch the Headgear Buckle and place the Headgear on your head like a headband with the Forehead Rest centered on your forehead in a comfortable position **Figure 6**.
2. Release both top left and top right Headgear tabs and gently pull on both tabs at the same time until the Headgear is comfortably snug. Do not overtighten.
3. Pull the bottom Headgear strap around the front of your face and connect the Headgear Buckle **Figure 7**.
4. Adjust the length of the bottom Headgear strap by pulling on either the bottom right tab or the bottom left tab so that the bottom strap will fit loosely on your chin **Figure 7a**.

Figure 6

Applying the Mask (360 Degree Option)

5. Remove the Quick Connect from the Mask Elbow and attach it to your air supply tubing.
6. Gently push the Mask Bridge into the opening on the Forehead Rest. **NOTE**—You may want to do this in front of a mirror the first few times.
7. Adjust the angle of the Mask on your nose using the Lower Mask Bridge Levers.
8. With your thumbs, pull the bottom strap of the Headgear up over the Mask Tusks **Figure 8**. By moving your thumbs back toward your ears, make sure that the bottom strap is flat against the mask body and your cheek. The Mask should now be snug but not tight. If adjustment to the bottom strap is required, lower the strap from the Mask Tusks and adjust the bottom right and/or bottom left headgear tab(s) length.
9. Turn on your CPAP device (refer to the Operating Instructions for your CPAP device), verify airflow through the air supply tubing, and attach the Quick Connect to the mask. You may feel a few small leaks around the Mask.
10. Eliminating mask leaks and optimizing comfort in the area around the bridge of the nose can be accomplished by adjusting the angle of the mask using the Lower Mask Bridge Levers.

CAUTION— Do not overtighten your headgear! Excessive tension will cause discomfort during the night. Reposition the mask on your face and adjust your headgear tension.

Figure 7

Figure 7a

Figure 8

Removing the Mask (360 Degree Option)

11. To remove the mask during the night, lower the bottom Headgear Strap from the Mask Tusks then press Upper Mask Bridge Levers and remove mask. If you prefer to leave mask attached, simply pull off the Mask Tubing Quick Connect from the Mask Elbow.

Figure 9 Over-the-Head Tubing Option

Over-the-Head Tubing Option (refer to Figure 9)

This option allows the air supply tubing to be fixed over the top of your head.

Applying the Headgear (Over-The-Head Option)

1. With the mask removed from the headgear, unlatch the Headgear Buckle and place the Headgear on your head like a headband with the Forehead Rest centered on your forehead in a comfortable position **Figure 10**.
2. Release both top left and top right Headgear tabs and gently pull on both tabs at the same time until the Headgear is comfortably snug. Do not overtighten.
3. Remove the Headgear and apply the Over-The-Head Strap by attaching part C to the back of the Headgear and parts A & B over the sides of the Headgear **Figure 11**.
4. Place the Headgear back on your head, check that it is comfortably snug, and adjust as necessary.
5. Pull the bottom Headgear strap around the front of your face and connect the Headgear Buckle **Figure 7**.
6. Adjust the length of the bottom Headgear strap by pulling on either the bottom right tab or the bottom left tab so that the bottom strap will fit loosely on your chin **Figure 10a**.

Figure 10

Applying the Mask (Over-The-Head Option)

7. Remove the Quick Connect from the Mask Elbow and attach it to your air supply tubing.
8. Pull down the Elbow Lock halfway and bring the Mask Elbow to the upright position. Using your two index fingers on either side of the Elbow, pull the Elbow Lock down to lock the Mask Elbow into place.
9. Gently push the Mask Bridge into the opening on the Forehead Rest. **NOTE**—You may want to do this in front of a mirror the first few times.
10. Adjust the angle of the Mask on your nose using the Lower Mask Bridge Levers.
11. With your thumbs, pull the bottom strap of the headgear up over the Mask Tusks **Figure 12**. By moving your thumbs back toward your ears, make sure that the bottom strap is flat against the mask body and your cheek. The Mask should now be snug but not tight. If adjustment to the bottom strap is required, remove the strap from the Mask Tusks and adjust the bottom right and/or bottom left headgear tab(s) length.
12. Turn on your CPAP device (refer to the Operating Instructions for your CPAP device), verify airflow through the air supply tubing and attach the Quick Connect to the mask by bringing the air supply tubing over the back of your head. Then insert the air supply tubing into the Velcro loop **Figure 11-D** on the top of the headgear by releasing the Velcro first then bring the loop back together over the tubing. You may feel a few small leaks around the Mask.
13. Eliminating mask leaks and optimizing comfort in the area around the bridge of the nose can be accomplished by adjusting the angle of the Mask using the Lower Mask Bridge Levers.

Figure 10a

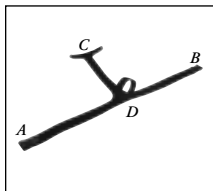
Figure 11

Figure 12

CAUTION— Do not overtighten your headgear! Excessive tension will cause discomfort during the night. Reposition the mask on your face and adjust your headgear tension.

Removing the Mask (Over-The-Head Option)

14. Release the upper Velcro Loop that holds the tubing and lower the bottom Headgear Strap from the Mask Tusks, then press Upper Mask Bridge Levers and remove mask. If you prefer to leave mask attached, simply pull off the Mask Tubing Quick Connect from the Mask Elbow after releasing the Velcro Loop.

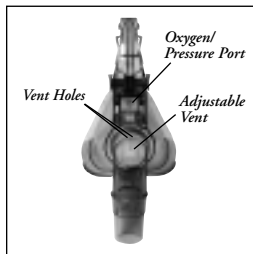
Figure 13

ADJUSTABLE VENT

The DeVilbiss FlexSet Mask has a patented feature that allows you to select the direction of the airflow exiting from the exhaust ports. The direction of the airflow can be adjusted by turning the Adjustable Vent from one side to the other **Figure 13**.

OXYGEN/PRESSURE PORT

This is for monitoring pressure during a sleep study or adding oxygen if prescribed by your physician. Simply remove the oxygen/pressure port cover **Figure 13** and secure your tubing to this port.

⚠ WARNING

Oxygen supports combustion. To avoid possible physical injury, do not smoke while using your CPAP with supplemental oxygen. Do not use near hot sparking objects or open sources of flame.

Figure 14

MASK CUSHION REMOVAL AND REPLACEMENT

The FlexSet Mask has two sizes of cushion: standard and shallow. Both sizes are available in either silicone or gel options.

To remove the cushion:

1. Gently pull the cushion away from the mask body.

To replace the cushion:

1. Place the mask cushion onto the mask body by first aligning the alignment mark on the top of the cushion with the alignment on the mask body, **Figure 15**. Next, push the top area of the mask cushion onto the mask body, **Figure 16**. Continue attaching the cushion to the mask body along the sides and bottom until the cushion is completely secured, **Figure 17**. If necessary, adjust the position of the cushion to eliminate any wrinkles in the cushion.

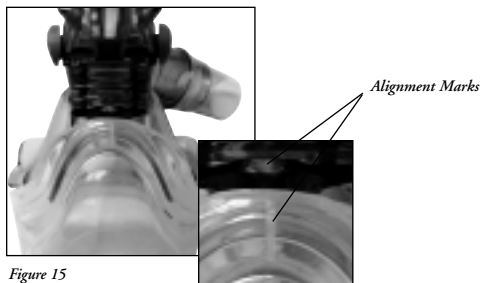
Figure 15

Figure 16

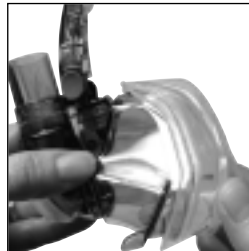
Figure 17

CARE AND CLEANING INSTRUCTIONS

Daily: Handwash. To disassemble the mask for cleaning, detach the 22mm diameter air tubing, headgear, mask cushion, forehead pad, and adjustable vent.

The mask and components can be washed in a solution of warm water and mild soap (e.g. Dawn® Dishwashing Liquid). Rinse the mask and components completely and allow to air dry before reusing.

CAUTION—Do not use bleach, chlorine-, alcohol-, or aromatic-based solutions, including scented oils, to clean the nasal mask assembly or air tubing. These solutions may cause hardening and reduce the life of the product. Do not place the tubing in direct sunlight, as it may harden and crack.

Weekly (if desired): The mask assembly is dishwasher safe if placed in the top rack of the dishwasher. It is recommended that the mask be washed separately and that all small components be secured. Allow mask to air dry before use.

To reassemble the mask after cleaning:

Insert the Adjustable Vent into the mask body opening from the outside. Ensure that the flange of the vent is fully inserted into the opening and that the vent is seated flat on the outside surface of the mask body. The Adjustable Vent should rotate easily within the opening.

Replace the mask cushion onto the mask body by first aligning the alignment mark on the top of the cushion with the alignment mark on the mask body, **Figure 8**. Next, push the top area of the mask cushion onto the mask body, **Figure 9**. Continue attaching the cushion to the mask body along the sides and bottom until the cushion is completely secured, **Figure 10**. If necessary, adjust the position of the cushion to eliminate any wrinkles in the cushion.

Reattach the forehead pad to the forehead rest. Reassemble the headgear to the mask as instructed in either the *360 Degree Option* or *Over the Head Option* sections of this guide.

The mask should be inspected before every use to insure that the vent holes are free from obstructions **Figure 13**. Replace your nasal CPAP mask if it develops cracks or tears. **NOTE**—For optimum mask performance, it is recommended that you replace your mask at least once a year. See your home medical equipment provider for replacement information.

WARNING

This nasal CPAP mask is intended for single-patient use only. The mask may be cleaned and reused on the same person. The mask must not be used on more than one person.

WARNING

This mask should not be used unless a positive airway pressure device is attached and operating. The vent holes (Figure 13) in the mask should never be blocked. The flow of air from a positive airway pressure device is needed to flush the exhaled air from the mask. If the positive airway pressure device is not operating, exhaled air may be rebreathed.

WARRANTY

DeVilbiss FlexSet Mask is warranted to be free from defective workmanship and material(s) for a period of six months from date of purchase. Any defective part(s) will be repaired or replaced at Sunrise Medical's option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your Sunrise Medical provider or Sunrise Medical Service Department for instructions:

U.S.A. 800-333-4000, 1-814-443-4881

Canada 1-800-263-3390

Europe 44-138-444-6688

NOTE– This warranty does not cover providing a loaner unit, compensating for costs incurred in rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

DECLARATION OF CONFORMITY

Manufacturer: Sunrise Medical
Address: Respiratory Products Division
100 DeVilbiss Drive
Somerset, Pennsylvania 15501-2125 USA

Product Designation: Nasal CPAP Mask

Type, Model: DeVilbiss® FlexSet™ Mask, 9354 Series

We herewith declare that the above-mentioned product complies with the requirements of EC Directive 93/42/EEC and the following:

Class: IIa Rule 2

Quality System Standards Applied: ISO9001/ISO13485

Notified Body RWTÜV

MDD Annex II Applied

Authorized Representative: Sunrise Medical Ltd.
Sunrise Business Park
High Street
Wollaston, West Midlands DY8 4PS
ENGLAND
44-138-444-6688

®Registered U.S. Patent and Trademark Office and other countries.

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®Déposé auprès du U.S. Patent and Trademark Office et dans d'autres pays.
®Eingetragen beim dem U.S. Patent and Trademark Office sowie in anderen Ländern.
®Gedeponneerd bij U. S. Patent and Trademark Office en in andere landen.
Geautoriseerde vertegenwoordiger: Sunrise Medical Ltd.