

## DeVilbiss Mask® Cleaning & Disinfection Guide



**Note: Clean the mask prior to using the disinfection processes.**

### **Cleaning Procedure Prior to Disinfection**

1. Prior to cleaning, disassemble the mask according to the Instruction Guide:
  - a. DeVilbiss Serenity® Nasal CPAP Mask – A-9352
  - b. FlexSet® Nasal CPAP Mask – A-9354
2. Prepare a mild detergent (e.g., Dawn®) according to the manufacturer's recommendations in a clean plastic pan. Use cool-to-lukewarm water to prepare the detergent.
3. Immerse the mask completely in the prepared detergent, and allow it to soak for a minimum of one minute.
4. After the one-minute soak period, clean the mask using a soft bristle brush while the mask is still immersed to remove any visible soil material. While cleaning, pay particular attention to any crevices or folds on the mask.
5. After cleaning the mask and its parts, rinse in deionized water for a minimum of one minute.
6. Allow the mask to completely air dry before use.

## Pasteurization Procedure

1. Create a circulating water bath filled with sterile purified water (PURW) to 70°C (158°F).
2. Immerse the mask in the water bath and expose for 30 minutes.
3. Remove the mask from the water bath and air dry.

## High Level Disinfection

1. Prepare a disinfectant compatible with Polycarbonate and Silicone according to the manufacturer's recommendations.
2. Immerse the mask completely, filling all areas and eliminating air pockets.
3. Following the disinfection exposure, thoroughly rinse the mask by immersing it completely in a large volume of purified water. Allow the mask to remain in contact with the water for a minimum of one minute.
4. Remove the mask and repeat the rinse procedure a minimum of two additional times using fresh, purified water for each rinse.
5. After rinsing, allow the mask to completely air dry before use.

**NOTE:** Follow all manufacturer's instructions and guidelines for product use.

DeVilbiss Mask Cycle Tolerances		
DeVilbiss Mask	Cleaning	Pasteurization
Serenity Silicone (Medium 9352D and Small 9352S)	26	10
Serenity Gel (Medium 9352G and Small 9352GS)	26	10
FlexSet Silicone (Medium 9354D and Small 9354S)	26	10
FlexSet Gel (Medium 9354G and Small 9354GS)	26	10

Note: Velcro® patches may release from the FlexSet forehead pad with repeated disinfection.  
(Replacement Forehead Pad: p/n 9353D-604)



DeVilbiss Healthcare LLC | 100 DeVilbiss Drive | Somerset, PA 15501 USA  
800.338.1988 | 814.443.4881 | [www.DeVilbissHealthcare.com](http://www.DeVilbissHealthcare.com)