

## DeVilbiss SmartCode® Remote Therapy Monitoring

### Simple, Easy and Free!



DeVilbiss SmartCode  
as displayed on  
IntelliPAP

The DeVilbiss SmartCode is an effective and efficient way to remotely track adherence data. **With a simple phone call to the patient, providers or clinicians can obtain key adherence monitoring data.** Best of all, the **data is objective** as required by the new Medicare CPAP Local Coverage Determination (LCD).

The SmartCode feature is included in every DeVilbiss IntelliPAP® system **without additional cost or fees.**

### Performance Features

- A simple phone call to the patient to retrieve the SmartCode will satisfy the new CPAP LCD requirements
- No additional fees or equipment – SmartCode is built into each and every IntelliPAP System
- Monitoring CPAP adherence via SmartCode is simple to integrate into your existing CPAP patient follow-up protocol
- 1-day and 7-day codes allow you to contact the patient early in therapy to assess adherence and intervene if necessary.
  - Research has indicated the first two weeks of therapy are the most critical to establishing long-term adherence

### Now with Adherence Score

- The Adherence Score is a percentage of the best consecutive 30 days of usage that meet the CPAP LCD
  - For example, if within the last 30 days 22 of them were above the adherence threshold {4 hours or more of continuous use each night} and 8 days were below that adherence threshold, the Adherence Score would be 73.3% which would satisfy the CPAP LCD requirements for compliance
- Shows the best adherence percentage achieved for any rolling period of 30 consecutive days of use within a 90-day time frame
- The adjustable adherence threshold can be set to 4 or 5 hours with a default setting of 4 hours

## Reporting Features

**SmartCode** uses a proprietary encryption method that allows therapy and usage data to be generated and compressed into onboard codes. Much like a barcode, each character generated in the SmartCode represents a data set. When the SmartCode generated by the IntelliPAP is entered into the SmartCode report generator, the series of characters is deciphered into usable data.

SmartCode provides summary data for pre-defined time frames, each representing a 'rolling window' of time.

There are two SmartCodes generated by the IntelliPAP Standard:

- SmartCode: (One report for 1, 7, 30 and 90 days and Adherence Score) Each time frame reports the following parameters:
  - % of Days at Least 4 or 5 Hours
  - Day Count
  - Days at Least 4 or 5 Hours
  - Average While Breathing Hours Per Day

	IntelliPAP Standard	IntelliPAP Standard Plus, AutoAdjust, Bilevel S and AutoBilevel
<b>Available SmartCode</b>		
SmartCode (includes Adherence Score)	x	
Smart Code Usage	x	x
1-day		x
7-day		x
30-day		x
90-day (includes Adherence Score)		x
<b>Parameters Reported</b>		
Days at Least 4 hours	x	x
% of Days at Least 4 hours	x	x
Day Count	x	x
Average Hours per Day While Breathing Hours	x	x
Percentile Pressure		x
Apnea/Hypopnea Index (AHI)		x
Pressure Plateau		x
High Leak Flow Time		x
Non-Responding Event Index (NRI)		x
Exhale Puff Index (EPI)		x

For complete definitions of terms visit [www.intelliPAP.com](http://www.intelliPAP.com) under the Help button

- SmartCode Usage: Provides cumulative data since the start of therapy
  - Total While Breathing Hours (up to 4 years)
  - Total Days of Use (up to 3 years)

There are five SmartCodes generated by the IntelliPAP Standard Plus, AutoAdjust, Bilevel S and AutoBilevel:

- 1-Day SmartCode: Provides data for the last day of usage
- 7-Day SmartCode: Provides a summary of the last 7 days of usage
- 30-Day SmartCode: Provides a summary of the last 30 days of usage
- 90-Day SmartCode: Provides a summary of the last 90 days of usage plus Adherence Score
- SmartCode Usage: Provides cumulative data since the start of therapy
  - Total While Breathing Hours (up to 4 years)
  - Total Days of Use (up to 3 years)

### Usage Threshold

The minimum amount of time the patient must use the device to be logged as a compliant day – can be set to 4 or 5 hours with default of 4 hours

### Days at Least X Hours

Number of days where total usage is at least X hours a day based on Usage Threshold

### % Days at Least X Hours

Percentage of days where the total usage within each day is at least X hours based on Usage Threshold

### Day Count

Number of days measured within a timeframe i.e. if a 90-day code is collected at day 45 of therapy Day Count would be 45 days

### Average Hours per Day While Breathing Hours

Total number of hours user has been breathing off the device

### 95th Percentile Pressure

AutoAdjust pressure was at or below this pressure 95% of the time

### 90th Percentile Pressure

AutoAdjust pressure was at or below this pressure 90% of the time

### AHI

Apnea/Hypopnea Index represents the average number of apneas and hypopneas per hour

### Pressure Plateau Time

Percentage of time the AutoAdjust spent at the upper pressure setting

### High Leak Flow Time

Percentage of time where leak was above 95 liters per minute

### NRI

Non-Responding Event Index represents the average number of non-responding event that is detected but by design does not result in a pressure change) events per hour.

### EPI

The Exhale Puff Index is the number of exhale puff events detected per hour

## Design Features

### Using SmartCode for Medicare Compliance

SmartCode is simple and efficient for both you and your patients. Just follow these three steps:

1. Call the patient and check their Adherence Score
2. If Adherence Score is acceptable, collect SmartCode
3. Use the online report generator to document adherence and therapy data

### Easy to Use

Call the patient and walk them through the simple steps below.

### IntelliPAP Standard Plus, AutoAdjust, Bilevel S and AutoBilevel

1. Make sure IntelliPAP is plugged in
2. Push the left-arrow twice
3. Read Adherence Score  
(If Adherence Score is above 70% proceed with next steps, otherwise coach the patient and schedule a follow-up call)
4. Push the right-arrow once
5. Push the down-arrow three times until the 90-day code is displayed and read the code (optionally collect the other codes for comprehensive data)

### IntelliPAP Standard

1. Make sure IntelliPAP is plugged in
2. Push the left-arrow twice
3. Read Adherence Score  
(If Adherence Score is above 70% proceed with next steps, otherwise coach the patient and schedule a follow-up call)
4. Push the right-arrow once
5. Read the SmartCode

Go to [www.intellipap.com](http://www.intellipap.com) and enter the code(s) to generate a report.



### SmartCode Test Drive

Try out SmartCode today by visiting [www.intellipap.com](http://www.intellipap.com) and entering one or more of the following codes:

### IntelliPAP Standard Sample Codes

- SmartCode: 16T-TCW5-2FK
- SmartCode Usage: ZDN-79FK-KFN

### IntelliPAP Standard Plus Sample Codes

- 1-Day: FJ4F-3B3-3P2Q
- 7-Day: 3CCC-379D-7C20
- 30-Day D0Y4-3D97-DP2L
- 90-Day: 7MTJT-4594-7DV2P
- SmartCode Usage: ZD3-79FK-K2V

### IntelliPAP AutoAdjust Sample Codes

- 1-Day: F6CD-37F-357R
- 7-Day: 3CCC-377D-7J7X
- 30-Day: DWY4-3DB7-DJ7L
- 90-Day: 752TT-47DB-7DT7C
- SmartCode Usage: ZDN-79FK-K7H

### IntelliPAP Bilevel S Sample Codes


- 1-Day: 46VV-37F-3AEH
- 7-Day: ACCV-V77D-76EJ
- 30-Day: 5WYV-VDB7-D6EQ
- 90-Day: H52TT-VVDB-7DREL
- SmartCode Usage: ZDN-79FK-KEZ

### IntelliPAP AutoBilevel Sample Codes

- 1-Day: B6CD-39F-35M1
- 7-Day: 9CCC-379D-7DM3
- 30-Day: EXY4-3D97-DJMN
- 90-Day: CMTJT-47D9-7DTM4
- SmartCode Usage: ZD7-79FK-KMJ



**IntelliPAP AutoAdjust Summary Report Using SmartCode System**

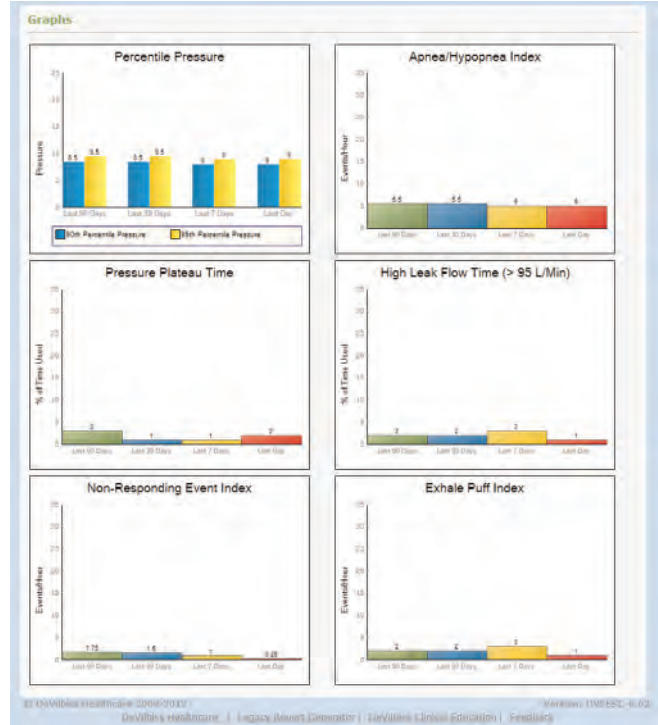

**IntelliPAP AutoAdjust Bar Graph Reports Using SmartCode System**

- Easy-to-use web-based report generator
  - HIPAA Compliant – no patient data is saved on our web servers. The website serves as a SmartCode decoder and allows you to generate and save or print reports
  - Usage data reported by SmartCode meets the CPAP LCD requirements
- Reports display data graphically and statistically

## Technical Specifications

**Internet Connection Required**

**Web Browser Compatibility:**

**IntelliPAP Compatibility:**

Microsoft® Internet Explorer®, Mozilla®, Firefox®, Apple® Safari®

IntelliPAP Standard, IntelliPAP Standard Plus, IntelliPAP AutoAdjust, IntelliPAP Bilevel S, IntelliPAP AutoBilevel



DeVilbiss Healthcare LLC | 100 DeVilbiss Drive | Somerset, PA 15501 USA  
800.338.1988 | 814.443.4881 | www.DeVilbissHealthcare.com