


Follow these instructions to properly position your Innova headgear and mask. The device must be turned on in order for the mask to seal to the face properly. There should be minimal or no leaks. **NOTE:** Refer to instruction manual for more detailed instructions.

STEP 1

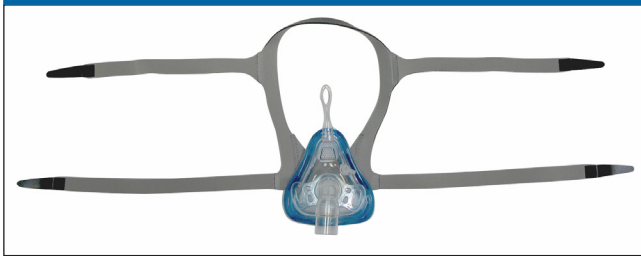


Remove the mask and the headgear from the package. Place the headgear on a table, label side down with the narrowest part close to you. Set the mask with the gel side down onto the center of the headgear.

STEP 2




Take each headgear strap and connect it to its respective location on the mask by looping the Velcro strap through the headgear slots and attaching it back to itself.

STEP 3

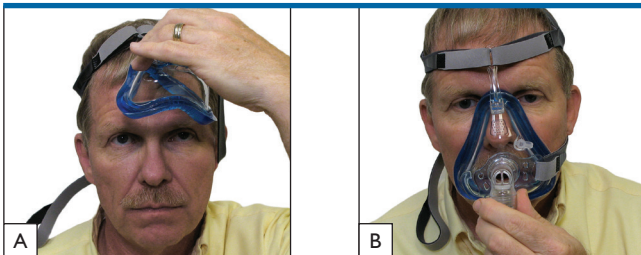

Detach one Velcro strap by sliding the Velcro loop off the quick release hook, slip the headgear over the head and center the mask on the face.


STEP 4

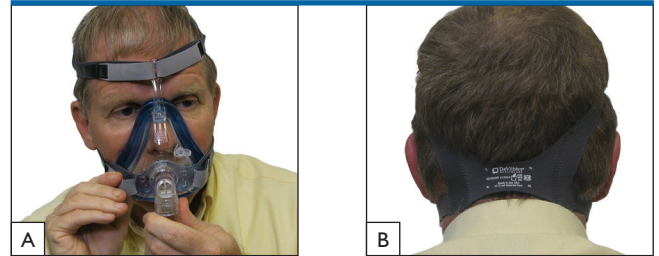

Find the detached Velcro strap and slide the Velcro loop back on the quick release hook.


Be sure the headgear straps are parallel and the headgear is fully extended and flat across the back of the head.

STEP 5

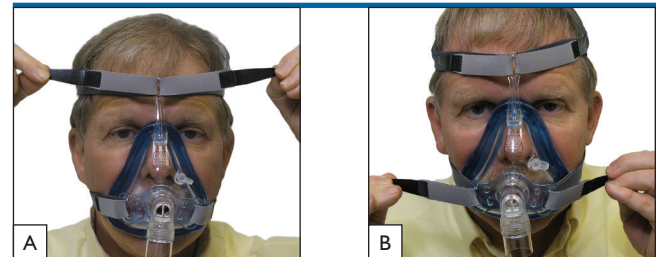

Adjust the top and bottom headgear straps by using the Velcro tabs. Make sure the Active Headgear Connector™ is parallel to the face and that there is equal tension from the headgear on both sides. After tightening, make sure the Active Headgear Connector can move freely.

STEP 6

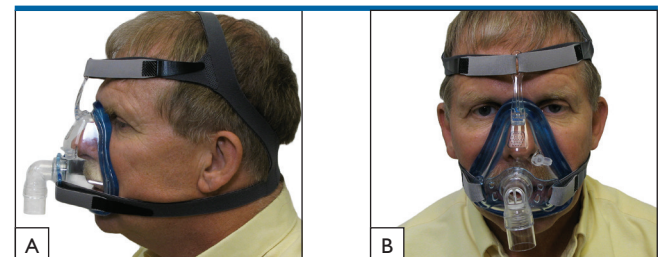

Ensure the flexible spacebar is nearly parallel to the face and does not touch the forehead.

Test that the mask is in the proper position and seals well by connecting the tubing to the sleeve and setting the device to the prescribed pressure. There should be minimal or no leaks.