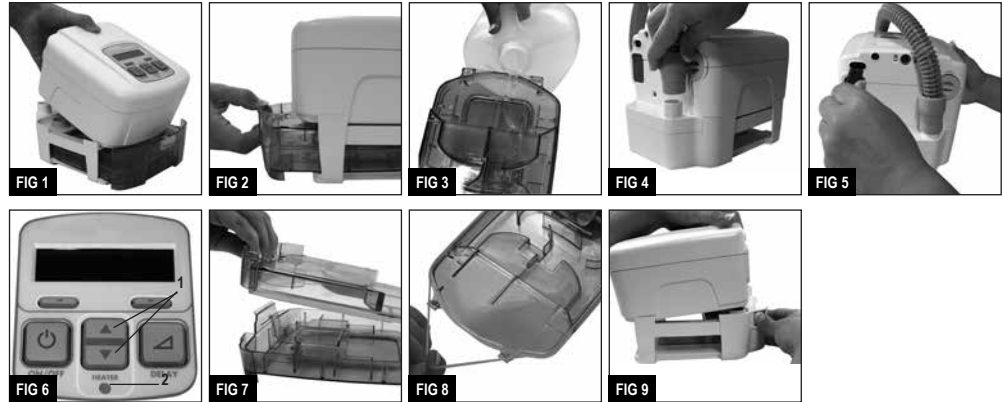
Assembly
Montaje
Assemblage
Anschluss
Assemblaggio
Montagem
Assemblage
Montering
Järjestelmän
Kokoaminen

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EN - ENGLISH

WARNING

- Do not attempt to fill the humidifier while it is attached to the humidifier cradle. Damage to the humidifier cradle may occur.
- If the system is placed on the floor, do not position it where it may be stepped on when getting out of bed.
- Never touch the heater plate on the humidifier cradle. Never touch the heat transfer plate on the bottom of the water chamber. These plates can reach temperatures as high as 149°F (65°C) during operation.
- Do not operate the heater if the water chamber is empty. The heater plate may be turned off using the keypad heater control when being used without water.

OPERATING INSTRUCTIONS

Heater Controls (Fig 6)

The heater controls and status are located on the keypad and display.

1. Heater Setting Control Buttons
2. Heater Power LED

Controlling the Humidity

The heater setting can be adjusted in a range of 1 to 10. The optimum temperature setting is dependent on the ambient temperature and humidity of your room. If your room temperature is low and/or the relative humidity is high, then a lower temperature setting may be necessary for optimum comfort. The heater setting of 1 results in a heater plate temperature of approximately 84°F (29°C). The heater setting of 10 results in a heater plate temperature of approximately 149°F (65°C).

Daily Use Instructions

NOTE– Always allow at least 10 minutes of cooling time after turning the device off before removing the water chamber from the humidifier cradle.

1. Press down on the chamber release latch of the humidifier and remove the chamber from the cradle (Fig 2).
2. Open the chamber and empty any water remaining from the previous night.
3. Clean the chamber according to the cleaning instructions and reassemble.

CLEANING INSTRUCTIONS

Humidifier Water Chamber

1. Allow device to cool down for at least 10 minutes. Press down on the chamber release latch and pull out the water chamber from the cradle (Fig 2).
2. Separate the chamber halves by pressing forward on the chamber disassembly lever and lifting the lid of the chamber (Fig 7).
3. Remove the sealing gasket from the chamber lid (Fig 8).
4. Wash the parts in a solution of warm water and mild dishwashing detergent.

CAUTION– Do not use any solutions that contain bleach, chlorine, alcohol, scented oils or other aromatic-based ingredients to clean the humidifier components or tubing. These solutions may reduce the life of the product.

5. Allow the parts to dry before reassembly.

CAUTION– Inspect the gasket seal before each use to ensure that it is properly seated in the groove around the entire chamber.

NOTE– The disassembled chamber may also be washed once per week in a dishwasher on the delicate or glassware cycle, top shelf only.

Humidifier Cradle

NOTE– The humidifier cradle should be cleaned as needed.

1. Turn off the heater and unplug from the power source. Allow the system to cool for at least 10 minutes.
2. To detach the flow generator from the humidifier cradle, first remove the water chamber, then fully depress the flow generator release button to separate (Fig 9).

NOTE– The flow generator release button must be pressed completely to separate.

3. Use a soft, damp cloth to clean the surfaces of the unit.

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