
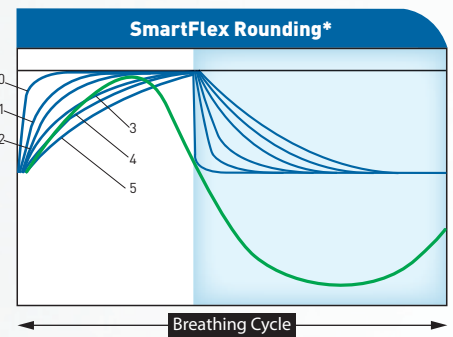
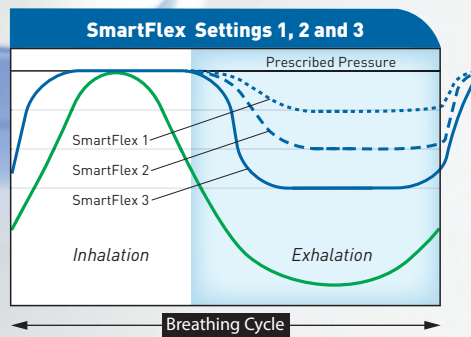


“With SmartFlex™, you and your patients will breathe easier.”

The IntelliPAP® AutoAdjust® and IntelliPAP® Standard Plus now include a unique exhalation pressure relief feature called SmartFlex. SmartFlex provides three pressure relief drops and six rounding settings to ease inhale and exhale transitions. The SmartFlex waveform can be “rounded” to make the pressure drop and the corresponding increase feel even more natural to each patient. SmartFlex offers many other features such as:

- ▣ Three selectable pressure relief levels
- ▣ Exact vs. relative pressure drop
- ▣ Increment of pressure relief levels of 1 cm H₂O
- ▣ Six independent inspiratory and expiratory transitions each
- ▣ Includes relief disengagement in the absence of sufficient breath (i.e. apnea)
- ▣ Can be used in delay-only or full-time mode

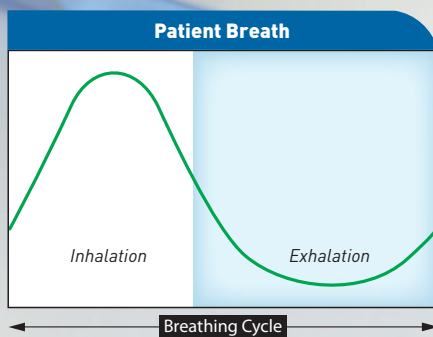

SmartFlex offers three pressure reduction settings that will automatically reduce the pressure generated by the IntelliPAP upon exhalation and raise the pressure back up to prescribed pressure at the onset of inspiration. The settings correlate to an exact cm H₂O pressure drop – so a setting of 1 = 1 cm H₂O drop in pressure during exhalation.

To enhance comfort, SmartFlex Rounding is a patented* customization of transitions between inhale and exhale. This allows a user to experience a more natural breathing waveform.

* U.S. patent 5,865,173.



DeVilbiss Healthcare LLC | 100 DeVilbiss Drive | Somerset, PA 15501 USA
800.338.1988 | 814.443.4881 | www.DeVilbissHealthcare.com