


VixOne™ Nebulizer

Highly efficient small-volume nebulizer

The cornerstone of our nebulizer line is the VixOne™ nebulizer. This is the economical choice in small-volume nebulizers. Its ideal particle size production **delivers maximum therapeutic effect** by increasing the amount of medication that is delivered deep within the lungs.

This **highly efficient nebulizer** is offered in both disposable and reusable models. These models are latex free and are designed for ease of use, allowing medication to be added without opening the neb cup while **standard fittings accommodate most manufacturers' masks**. The reusable nebulizer is top-shelf dishwasher safe.

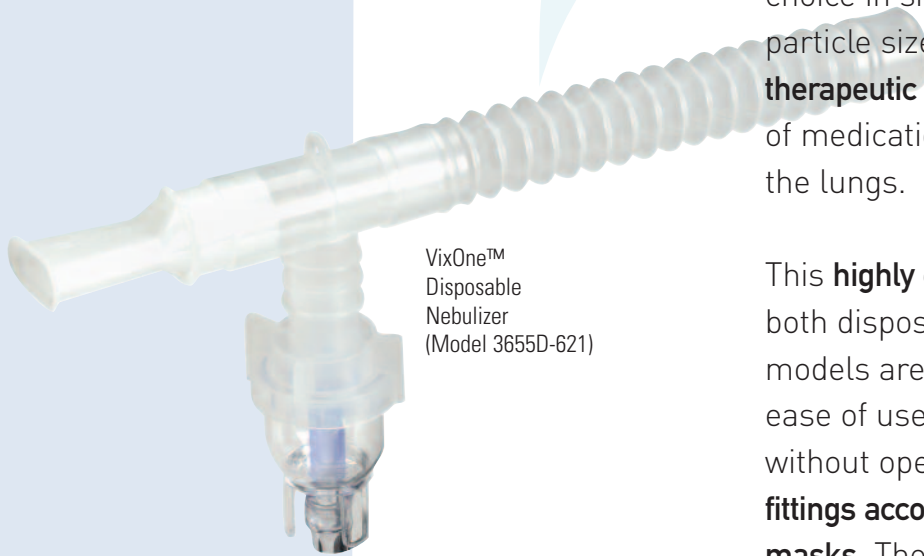


VixOne™
Disposable
Nebulizer
(Model 3655D-621)

Performance Features

- Produces an aerosol with an MMAD of 2.7 microns
- Respirable Fraction — 84% below 5 microns
- **At least 18% more medication delivered** than with a traditional open system
- **Low residual volume** (0.7cc) reduces medication waste
- High-capacity medication bowl (>10cc) accommodates varying therapy protocols
- Nebulizes in upright position or at angles up to 45°, allowing patients to receive treatment in their most comfortable position

VixOne and AeroEclipse® Nebulizer Performance with DeVilbiss PulmoNeb®

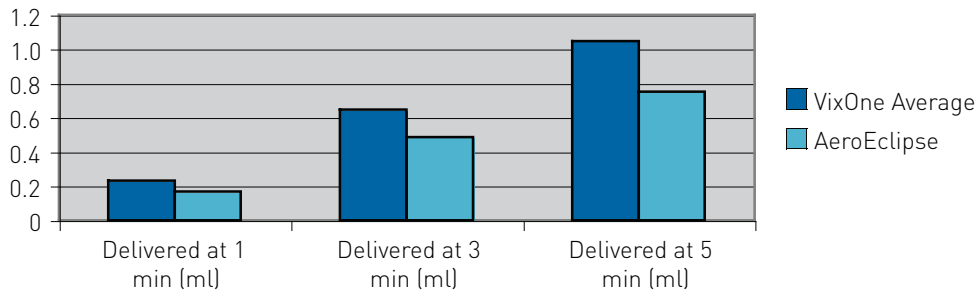

This graph depicts the performance characteristics for the VixOne Nebulizer when paired with the DeVilbiss PulmoNeb.

Design Features

- Comfortable mouthpiece design
- Universal connector accommodates standard adult and pediatric masks as well as the PediNeb™ pediatric pacifier mouth piece (see below for more information)
- **Anti-spill tee design** provides secure seal with simple ¼ turn assembly design
- 7' kink-resistant Flexhose tubing
- **Latex free** for safe use by those with latex allergies

PediNeb™ Pediatric Nasal Aerosol Pacifier

The PediNeb was specially designed to enhance the delivery of aerosolized medication to infants and toddlers while **reducing the apprehension experienced by young patients.**

- Pacifier calms the young child during aerosol treatment
- Medication port **directs aerosolized medication to the nose** and reduces medication waste associated with the “blow-by” technique
- **Easy to operate**, clean, and maintain
- Patented design was developed by a pediatric nurse practitioner
- Comes with VixOne for disposable nebulizer



Ordering Information

ITEM	PART NUMBER	QUANTITY
VixOne Disposable Nebulizer	3655D-621	Case of 50
VixOne Reusable Nebulizer	900D-610	Case of 10
VixOne Disposable Nebulizer with PediNeb	3655PN-612	Case of 12



DeVilbiss Healthcare | 100 DeVilbiss Drive | Somerset, PA 15501 USA
800.338.1988 | 814.443.4881 | www.DeVilbissHealthcare.com