

VixOne™ for DeVilbiss

REUSABLE REGULAR USE NEBULIZER WITH
MOUTHPIECE, TEE, CORRUGATED TUBING AND 7' SUPPLY TUBE

Manufactured for:



Somerset, PA 15501-2125 USA
www.devilbisshealthcare.com
800.338.1988



016958940398

ORDER #900D-610
UNITS/CASE - 10



CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician. Single patient use. Not intended for reprocessing.

Instructions for Use

1. This product is intended for use in delivering aerosol medications.
2. Remove nebulizer from the package and add prescribed medication in opening at the top of nebulizer.
3. Attach corrugated tubing to tee, then attach tee to nebulizer top. Attach mouthpiece to open end of tee.
4. Attach 7' tubing to bottom of nebulizer cup, connect other end of tubing to your DeVilbiss pulmonary drug delivery device's air outlet port.
5. Ensure all connections are secure, turn on your DeVilbiss unit. Confirm appropriate aerosol function. If necessary, tap nebulizer cup until mist appears.
6. Begin treatment as instructed by your physician. Continue until all medication has been aerosolized. Some of the medication may cling to the sides of nebulizer cup during treatment. Occasionally tap cup to nebulize medication.
7. Can be used up to six months.



This product is designed for use with compressors; not tested for use with anesthetic or ventilator systems. Patients using oxygen therapy should never do so near ignition sources.

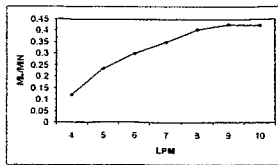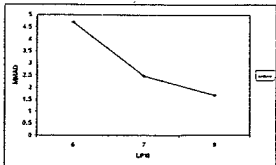

Performance information may not apply to drugs supplied in suspension or high viscosity form, consult drug supplier.

LOT N11745

Westmed

5580 S. Nogales Highway
Tucson, Arizona 85706 USA
Phone: 800-975-7987
Fax: 520-294-6061
www.westmedinc.com



MT Promedt Consulting GmbH
Altenhofstr. 80
66386 St. Ingbert, Germany
ASSEMBLED IN MEXICO
LABEL PN 71790, Rev. 01